



TytoCare Platform: Cleaning Instructions

COVID-19: In response to cleaning requirements, all eCARE equipment should be cleaned with standard 70% Isopropyl Alcohol.

eCARE has deployed the TytoCare platform with partnering facilities, instructions are as follows for cleaning the equipment.

- **Cleaning the Physical Cart**
 - a. Always disconnect the system from power before cleaning or servicing.
 - b. Do not spray liquids directly onto the cart when cleaning.
 - c. Standard hospital grade cleaning wipes used for medical equipment can be used on the cart. Allow the unit time to dry prior to powering up.
- **Cleaning the TytoCare Device, Peripherals & Case**
 - a. Please hold device & peripherals securely while cleaning.
 - b. Remove dust from the lens using a dry soft micro fiber cloth using light pressure.
 - c. All exterior surfaces of the TytoCare peripherals excluding the iPad, used for patient care are required to be disinfected with each use and can be wiped down using germicidal wipes. Avera eCARE recommends standard hospital grade cleaning wipes.
- **Cleaning the iPad**
 - a. Remove dust from the screen with a dry soft micro fiber cloth using light pressure.
 - b. Using LCD/Screen Cleaning wipes, gently remove oil, fingerprints, and grime from the screen surface working from the center outward.
 - c. In the event the iPad comes into contact with a contaminant, standard 70% Isopropyl Alcohol Wipes can be used in place of LCD/Screen Cleaning wipes.

TytoCare: Additional Reference Material

eCARE developed a condensed version of information within this guide. For additional information please access TytoPro's knowledgebase located at the following link: <https://support.tytocare/knowledgebase> and select the **TytoPro** link.

Knowledge Base - Home
TytoHome
TytoPro
TytoClinic

In-depth support in the following categories is available: Getting Started, Performing a Visit, Troubleshooting, Product Guides, and FAQs